




Clarity™

Esthetic Orthodontic Solutions

3M™ Clarity™ Ultra Self-Ligating Brackets

**Practice confidently.
Smile beautifully.**

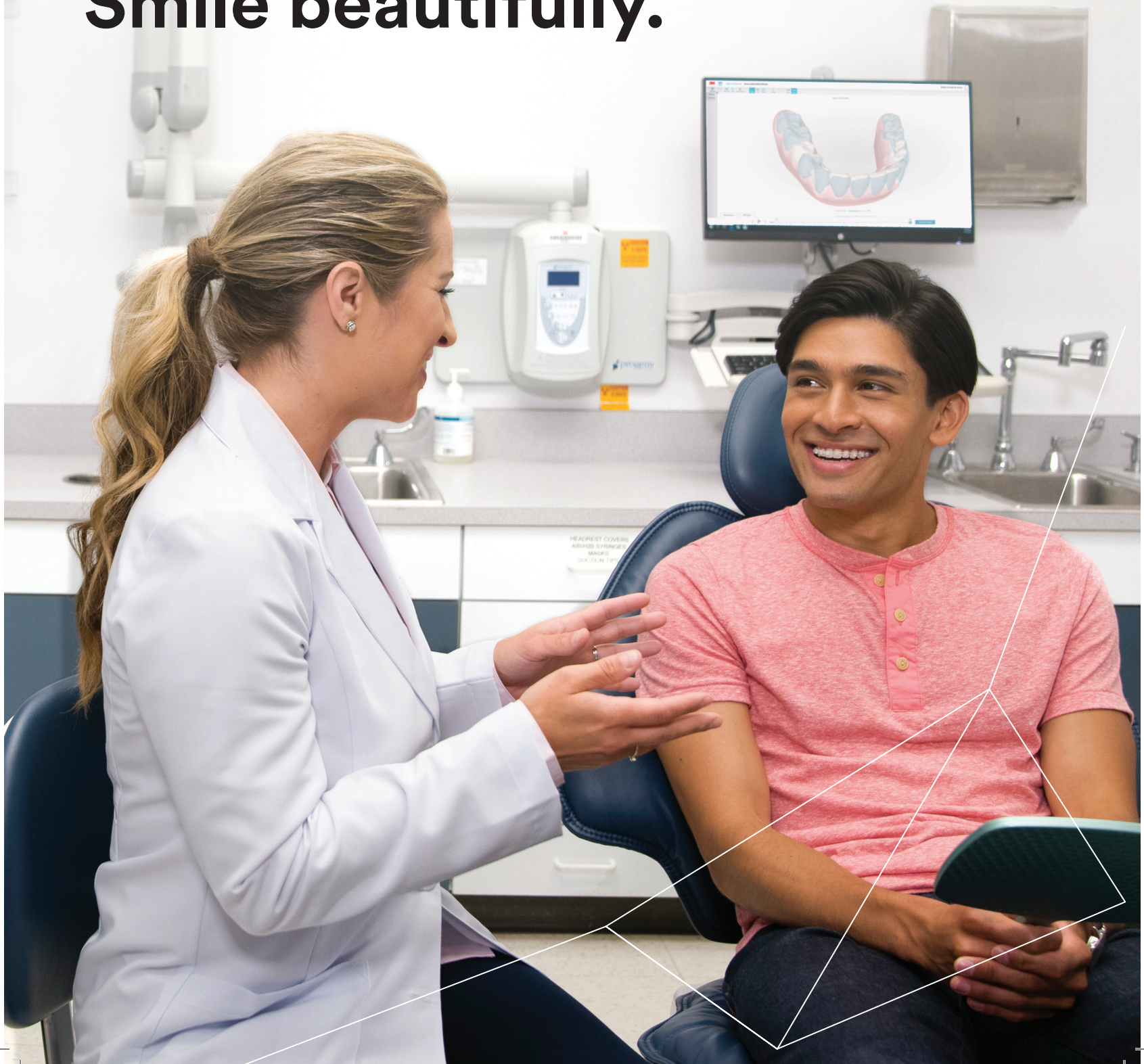




Creating success for patients and your practice.

As you know, the orthodontic business and treatment models are changing. Patients have more choices than ever, and they're playing an active role in decision making when it comes to the products they purchase. At the same time, emerging technology is driving innovation and new efficiencies in the practice of orthodontics.

In the midst of this change, your goal remains the same—creating beautiful, confident smiles that will last your patients a lifetime.

Showcase your expertise. Deliver precise results.

3M™ Clarity™ Ultra Self-Ligating Brackets offer rotational control, improved versatility and 3M's unique stress concentrator designed for predictable, consistent debonding.

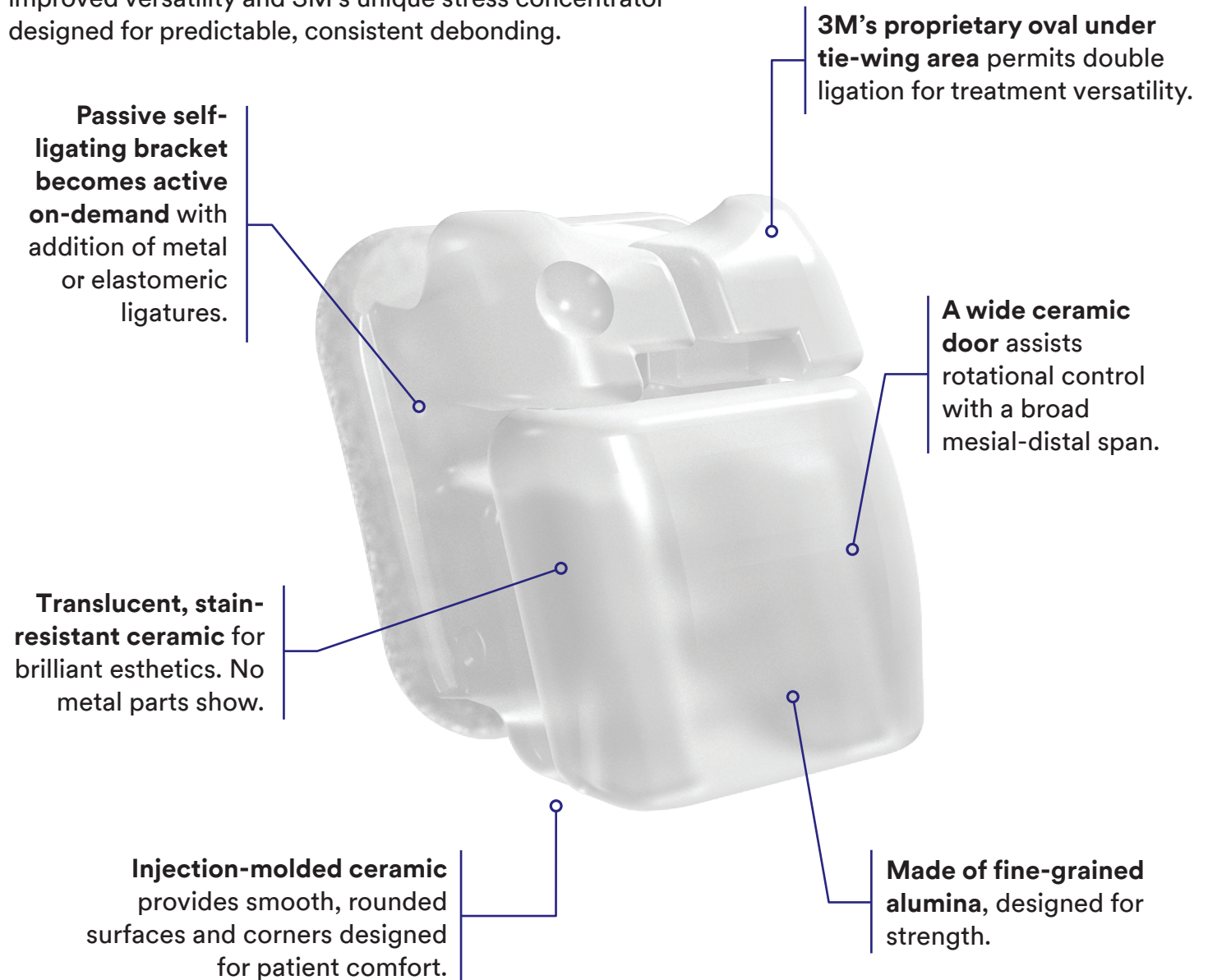

Doors withstand more than **2x** typical treatment use

Reliable, self-ligating doors withstand more than 2x the open- and closing- cycles of typical treatment.

Typical 2-year treatment

2x


3M™ Clarity™ Ultra Bracket door reliability

Empowering your efficiency and results.

You and your patients want the same thing: effective treatment options with positive results. To achieve this, you need control and predictability from start to finish. Clarity Ultra brackets give you that power—allowing for successful treatment planning to generate precise outcomes.



Easier wire changes
without ligation and
ligatures.

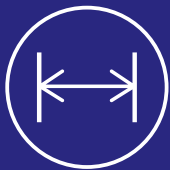

Wide ceramic door
that assists rotational
control with a broad
mesial-distal span.



Passive or active
on-demand.




Rotational control
without adding
accessories.



Simple, efficient bonding process.

Powered by technology, the 3M™ APC™ Flash-Free Adhesive Coated Appliance System is the most efficient bonding system in orthodontics. Our innovative design creates a unique adhesive bonding substrate on the base of the bracket that saves you time and positively impacts patient satisfaction.

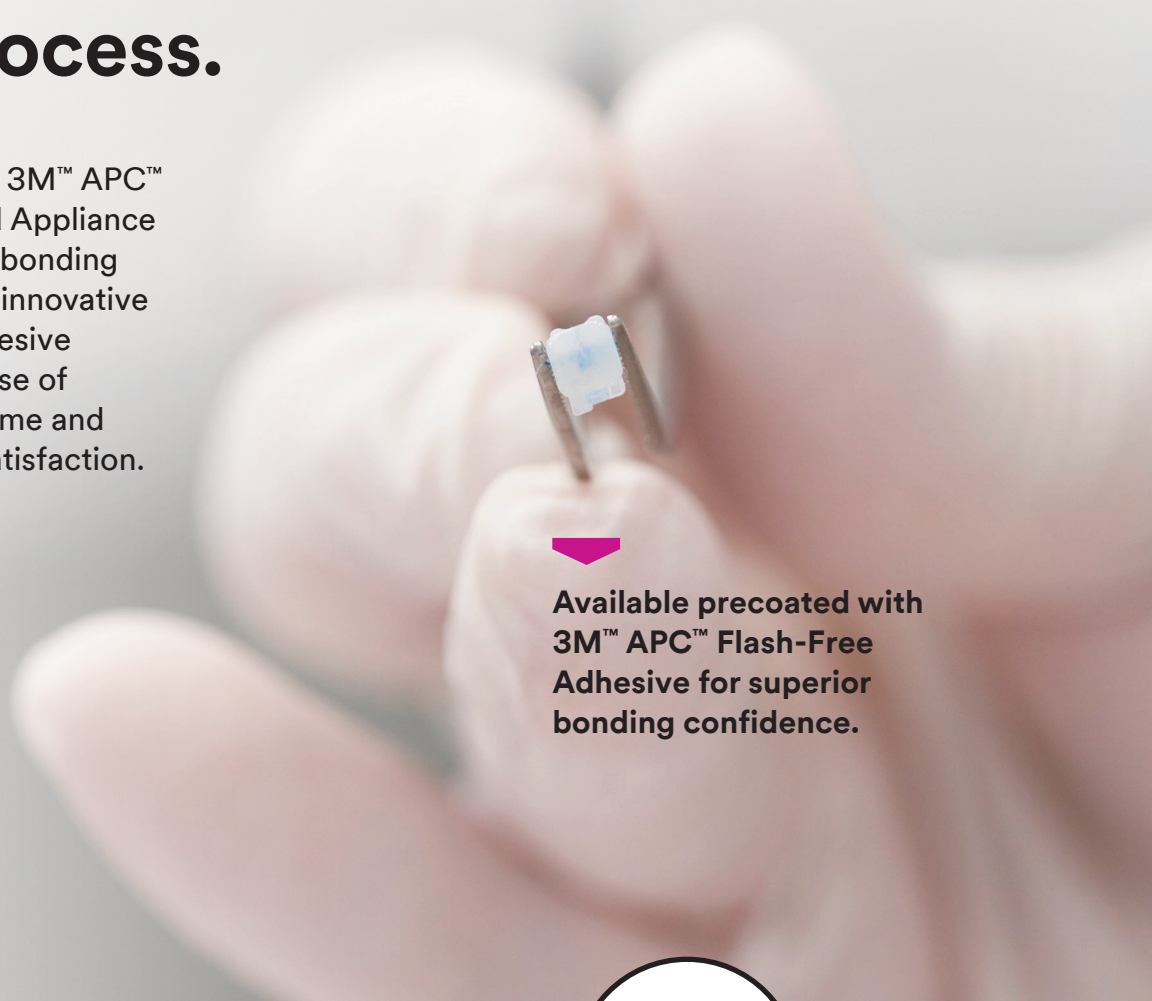

Available precoated with 3M™ APC™ Flash-Free Adhesive for superior bonding confidence.



Saves Time and Cost

- ✓ Faster and more efficient bonding than non-precoated brackets.
- ✓ Eliminates the flash removal step from bonding.
- ✓ Simple, easy clean-up at debonding.



Superior Bonding Efficiency

- ✓ Bond time is reduced up to 40% per bracket.
- ✓ No flash removal reduces accidental bracket movement after positioning, reducing the need to reposition.
- ✓ Protects tooth enamel under adhesive.

Driven by innovation you can count on.

When you choose the Clarity Ultra bracket system, you choose leading-edge technology, predictability and the customer service you rely on from 3M. More importantly, you have control of an esthetically-pleasing solution that performs throughout the treatment process and helps promote practice growth.



Maximizing efficiency

- ✓ Color-coded identification dots for simple bracket selection.
- ✓ Vertical and horizontal reference marks for easy bracket positioning on the tooth.
- ✓ Brackets shipped with the doors open, eliminating this step before placing the archwire.
- ✓ Unique stress concentrator designed for predictable squeeze debonding.



Performance reliability

- ✓ Door mechanism designed to resist unplanned opening and closing.
- ✓ Designed to provide an audible “click” sound when door is opened or closed.
- ✓ Doors last more than 2x the opening and closing cycles of typical treatment.



3M™ Clarity Ultra™
Self-Ligating Brackets



3M™ Victory Series™
Superior Fit Buccal Tubes



3M™ Forsus™
Class II Correctors

One methodology. Optimized practice efficiency.

Powered by 3M science, Clarity Ultra Self-Ligating Brackets and other 3M products – including APC Flash-Free Adhesive, 3M™ Forsus™ Class II Correctors and the 3M™ MBT™ Versatile+ Appliance System – are designed to work together to give orthodontists efficient and custom control to achieve results with a single treatment methodology.



A clear solution that inspires smiles.

Designed to blend in with natural tooth color, Clarity Ultra brackets provide a fully-esthetic solution. Translucent and stain-resistant, the bracket has no visible metal parts, giving it an attractive appearance.

The absence of elastomeric ligatures that may stain during treatment also enhances the visual appeal. Smooth, rounded ceramic surfaces and edges are designed to deliver comfort over the course of treatment that both you and your patients will appreciate.



3M™ Clarity™ Ultra Self-Ligating Brackets



3M™ MBT™ Appliance System Prescription

Maxillary			In/Out		Part Number - .022 in.	
Tooth	Torque	Angulation	in.	mm	L	R
Central	+17°	4°	0.026	0.66	003-301	003-302
Lateral	+10°	8°	0.035	0.89	003-303	003-304
Cuspid Hk	0°	8°	0.021	0.53	003-307	003-308
Bicuspid Hk	-7°	0°	0.029	0.74	003-310	003-311

Mandibular			In/Out		Part Number - .022 in.	
Tooth	Torque	Angulation	in.	mm	L	R
Anterior	-6°	0°	0.045	1.14	003-350	
Cuspid Hk	0°	3°	0.020	0.51	003-353	003-354
1 st Bicuspid Hk	-12°	2°	0.030	0.76	003-357	003-358
2 nd Bicuspid Hk	-17°	2°	0.034	0.86	003-361	003-362

Add Prefix "4" for 3M™ APC™ Flash-Free Adhesive System. Example: 4003-301

Patient Kits

	Part Number - .022 in.
U/L Lt/Rt 5×5 Hk	003-111
U Lt/Rt 5×5 Hk	003-116

Add Prefix "4" for 3M™ APC™ Flash-Free Adhesive System.

Instruments for the Ultra SL System

	Part Number
3M™ Unitek™ Debonding Instrument	804-175
3M™ Unitek™ Open-Close Instrument	804-155



REF 804-155



REF 804-175



Clarity™

Esthetic Orthodontic Solutions

Practice with clarity.

Our full portfolio of naturally-esthetic treatment options are informed by 3M science and fueled by creativity. Innovation drives our leading-edge technology and delivers the choice and control you seek.

Our commitment is to collaborate with orthodontists to provide custom, flexible esthetic treatment solutions that showcase your expertise and ensure precise outcomes. Together, we can change lives by creating healthy, beautiful smiles.



Visit [3M.com/ClarityUltra](https://www.3m.com/ClarityUltra), or ask your 3M Oral Care representative for more information.



3M Oral Care
2510 Conway Avenue
St. Paul, MN 55144-1000 USA
Phone 1-800-423-4588
Web [3M.com/ortho](https://www.3m.com/ortho)

3M Canada
300 Tartan Dr.
London, ON N5V 4M9 Canada
Phone 1-800-443-1661

3M, APC, Clarity, Forsus, MBT, Victory Series and Unitek are trademarks of 3M. Used under license in Canada.
© 3M 2020. All rights reserved.
70-2013-0965-8 2003