**Forsus™ Fatigue Resistant Device EZ2 Module –
Installation Guide**

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**Forsus™ Korrektor Module EZ2 –
Installationsvejledning**

3M

**3M Unitek
Orthodontic Products**
2724 South Peck Road
Monrovia, CA 91016 USA

EC REP 0086
3M Deutschland GmbH
Orthodontic Products
Carl-Schurz-Str. 1
D-41453 Neuss, Germany
+49(0)8191/ 9474-0

Forsus™ Fatigue Resistant Device EZ2 Module – Installation Guide

Instructions For Use

English

Indications for Use: This product is intended for use in orthodontic treatment.

Warning: This product contains nickel and/or chromium. A small percentage of the population is known to be allergic to nickel and/or chromium. If an allergic reaction occurs, direct patient to consult a physician.

Forsus EZ2 Module Installation

Holding the mesial portion of the Forsus™ Fatigue Resistant Device EZ2 Module with a pair of Weingart Utility Pliers, insert the Forsus EZ2 Module into the headgear tube from mesial to distal until it clicks into place. **(Note: Right and Left configuration denoted by “R” or “L” on each module.)**

Important: The Forsus™ EZ2 Module is fully compatible with all 3M Unitek Buccal Tubes with an occlusal headgear tube measuring 3.6 mm - 4.3 mm in length.

For all other buccal tubes, before installation of the Forsus EZ2 Module, ensure that the module snaps completely into the occlusal headgear tube for best performance.

Cinch archwire to secure lower arch teeth positioning. For increased anchorage on lower arch, use second molar attachment if appropriate. **(Note: Use full size rectangular stainless steel wire on both arches to further secure teeth positioning).**


In order to avoid elastic ligature breakage and to secure cuspid bracket to archwire, use stainless steel ligature tie.

Push Rod Installation

Measure from distal end of upper molar tube to the chosen mandibular stop while in centric occlusion. Use Measurement Guide (REF 807-014) as shown in picture and note appropriate push rod size (in this example, note 32 mm).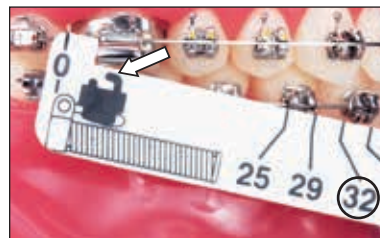

Select a push rod from the available sizes to fully compress the spring without immediately reposturing the mandible. **(Note: Right and Left configuration).**

Patient Right	Patient Left
	22 mm
	25 mm
	29 mm
	32 mm
	35 mm
	38 mm

Push Rod Installation, continued

Place push rod loop on archwire, have patient open mouth, compress spring, and insert push rod. If push rod protrudes distal of spring module in centric occlusion, it is too long.

Note: Attach loop over archwire from occlusal.


Note: Spring assembly length after installation should be 20-22 mm.

Crimp push rod mesial end by closing loop around archwire.


Re-Activation

Reactivation may only be achieved on push rod sizes that have stops: 25 mm, 29 mm, 32 mm, and 35 mm. To re-activate, crimp split ring bushings on push rod distal of stop by compressing the spring as needed (usually 2 or 3 mm at a time to achieve midline correction and advancement).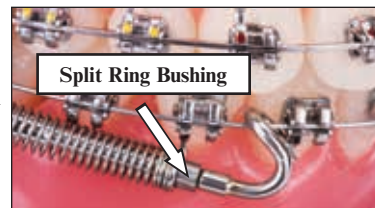

Forsus EZ2 Module Removal

First, with the patient's mouth open wide, compress the spring and remove the spring assembly from the push rod. Then, holding the distal portion of the Forsus EZ2 Module with a pair of Weingart Utility Pliers, pull the Forsus EZ2 Module in an occlusal/mesial direction to remove it from the headgear tube.

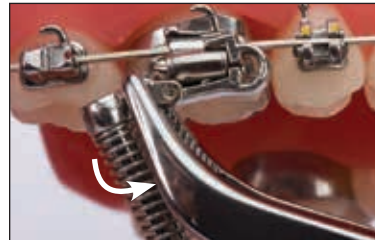Grasp the distal end of the push rod with your fingers. Holding the mesial loop in place with your Weingart plier, bend the push rod to open the mesial loop.

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