Forsus™ Fatigue Resistant Device –
Installation Guide

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Forsus™ Fatigue Resistant Device –
installationsvejledning

3M

3M Unitek
Orthodontic Products
2724 South Peck Road
Monrovia, CA 91016 USA

EC REP **CE** 0086
3M Deutschland GmbH
Orthodontic Products
Carl-Schurz-Str. 1
D-41453 Neuss, Germany
+49(0)8191/ 9474-0

Forsus™ Fatigue Resistant Device – Installation Guide

Instructions For Use

English

Indications for Use: This product is intended for use in orthodontic treatment.

Warning: This product contains nickel and/ or chromium. A small percentage of the population is known to be allergic to nickel and/or chromium. If an allergic reaction occurs, direct patient to consult a physician.

Spring Module Installation

Insert “L” ball pin into spring module distal-end pin hole.

Pull “L” ball pin through headgear tube from distal to mesial.


Bend pin around headgear tube as shown.

Option 1: pin bent occlusally.

Option 2: pin bent gingivally.

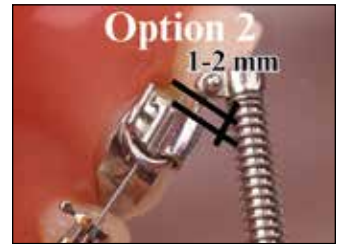(Note: To allow for pin adjustments and avoid restricted movement, leave 1 to 2 mm clearance between distal end of tube and pin ball as shown in picture).

Cinch archwire to secure lower arch teeth positioning. For increased anchorage on lower arch, use second molar attachment if appropriate.

(Note: Use full size rectangular stainless steel wire on both arches to further secure teeth positioning).

In order to avoid elastic ligature breakage and to secure cuspid bracket to archwire, use stainless steel ligature tie.

Optional

To avoid contact with cuspid bracket, place a stop such as a Gurin lock distal of the canine.

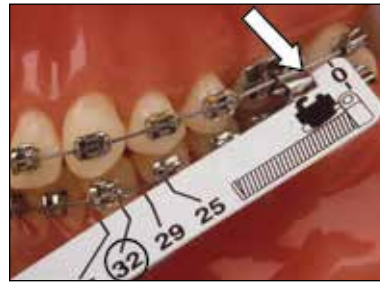
To increase anchorage on the lower arch and prevent lower anterior flaring, lace the entire lower arch and increase labial root torque on lower anteriors.

Push Rod Installation

Measure from distal end of upper molar tube to a point distal to bracket (or to chosen stop) while in centric occlusion. Use Forsus™ Device Gauge (REF 807-014) as shown in picture and note appropriate push rod size (in this example, note 32 mm).

Select a push rod from the available sizes to fully compress the spring without immediately reposturing the mandible. **(Note: Right and left configuration).**

| Patient Right | Patient Left |
|---------------|--------------|
| | 22 mm |
| | 25 mm |
| | 29 mm |
| | 32 mm |
| | 35 mm |
| | 38 mm |

Place push rod loop between cuspid and first bicuspid, have patient open mouth, compress spring, and insert push rod. If push rod protrudes distal of spring module in centric occlusion, it is too long.

Note: Attach loop over archwire from occlusal.

Note: Spring assembly length after installation should be 20-22 mm.

Crimp push rod mesial end by closing loop around archwire.

Final Assembly

Note: If patient bites on spring assembly, adjust "L" ball pin or entire assembly with a buccal offset

Re-Activation

Crimp split ring bushings on push rod distal of stopper by compressing the spring as needed (usually 2 or 3 mm at a time to achieve midline correction and advancement).

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