

Instructions for Use

Product's name: Headcap



Savaria-Dent Kft
H-9700 Szombathely
Dr. Szaboles Z. u. 5/A.
HUNGARY
Tel.: +36 94/505-840



Application

Function: A class I medical device that when connected to an orthodontic appliance, serves as a support for pulling.
Max. 2.5kg safety module release force load capacity per side.

Indications

To be used during orthodontic treatments based on the doctor's recommendation.

Contraindications


Allergic reactions while wearing the headcap during treatments.

Material: 100% cotton

Washing under 40°C with general detergent used for cotton.

The product cannot be used again once there is damage in the material, or along the seam.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

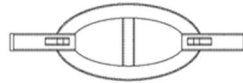

The safety module's connection

Version: 3.0 – 01.07.2021

Headcap

Product list

<u>Reference code</u>	<u>Trade name</u>
SD 48140	<u>Headcap, narrow ear</u>
SD 48141	<u>Headcap, wide ear</u>
SD 48142	<u>Headcap, narrow ear (middle)</u>
SD 48197	<u>Jeans headcap, wide ear</u>
SD 48198	<u>Jeans headcap, narrow ear</u>
SD 48198-O	<u>Jeans headcap, round ear</u>
SD 48199	<u>Jeans headcap, narrow 5 ears</u>