DIRECTIONS FOR USE

INTENDED USE:

Essix A+® Plastic is thermoformed to fabricate intra-oral appliances such as bite planes, splints, and snoring appliances. Essix A+® Plastic is suitable for use with non-bruxers/grinders.

INDICATIONS FOR USE:

Mouthguard and aligner materials are indicated for the fabrication of orthodontic and dental appliances such as aligners, bite planes, mouthguards, nightguards, snoring appliances, splints, retainers, repositioners, and temporary bridges.

INTENDED USER:

Intended to be used in a manufacturing setting within a dental laboratory, dental office or medical device manufacturing facility to fabricate a removable, custom-made intra-oral dental appliance.

PERFORMANCE CHARACTERISTICS AND CLINICAL BENEFITS:

Performance characteristics of dental plastic sheets include a good combination of strength, durability and clarity, resistant to cracking or breaking, easily removeable from models and effortless to trim.

Clinical benefits of orthodontic treatment may include overall improvement of tooth health and correction of malocclusion(s).

CONTRAINDICATIONS:

This product is contraindicated for patients with a history of allergic reactions to plastics.

WARNINGS:

Excessive heating time during thermoforming will cause plastic to over sag and potential for injury to operator.

Improper thermoforming of the plastic appliance may cause the appliance to crack/break, resulting in sharp edges, loose pieces and possible aspiration of pieces.

Report serious incidents that have occurred in relation to the device to device manufacturer and the Competent Authority in Member State of residence, or other applicable regulatory agency.

PRECAUTIONS:

Do Not Use if the foil package is open or damaged.

Essix A+® Plastic is not recommended for bruxers.

Patients with a history of allergic reactions to plastics should not use this product.

Do not reuse plastic sheet. Once thermoformed it cannot be reused.

Thermoformed device is for Single Patient Use. Risks of reuse include infection and contamination (i.e. bacterial, viral).

Reseal the foil bag when not in use to protect the plastic from environmental factors.

ADVERSE REACTIONS:

Allergic reactions.

Improper thermoforming of the plastic appliance may cause the appliance to crack/break, resulting in sharp edges, loose pieces and possible aspiration of pieces.

STEP BY STEP INSTRUCTIONS:

Thermoforming Instructions (for complete instructions, refer to thermoforming equipment manual):

- Essix A+® Plastic has a protective film on both sides. REMOVE FILM FROM BOTH SIDES BEFORE THERMOFORMING.
- All models should be trimmed to 3/4" high. Full arch models should have the palate trimmed away.
- For best results, we recommend spraying the model with Trim-Rite® or another biocompatible release agent to prevent the material from sticking to the model.
- Place the plastic into the frame of the machine and the model on the base plate.
- Scan/Enter the code per thermo-forming machine used. Swing the heating element over the plastic and press the appropriate start button.
- Move the heating element away from the plastic when the heating and cooling time expires.
- Drufomat: hold the element away and press the button on the side to engage the pressure chamber. Biostar: flip the heated plastic over the model.
- After the cooling stage, release the pressure per thermo-forming machine used.
- Remove the formed tray from the frame of the machine.
- Trim and finish the appliance as desired.

***NOTE:** The heating times below should be used as a reference as heating times may vary depending upon machine usage. If plastic does not adapt to model, add heating time until adaptation occurs. If plastic "folds" when thermoformed, reduce heating time until material does not fold. Performance with other machines have not been validated.

Thickness	Biostar™ Code	Drufomat
0.5mm (0.020")	122	1:00
0.75mm (0.030")	132	1:00
1.0mm (0.040")	142	1:05
1.5mm (0.060")	164	1:25
2.0mm (0.080")	205	1:30
3.0mm (0.120")	226	2:10

CLEANING:

Formed trays may be cleaned before each insertion by the patient with a dental appliance cleaner. Retainer Brite® products are recommended to clean trays formed with Essix® Plastic.

STORAGE:

Keep Dry

SAFE DISPOSAL:

Unused plastic sheets can be recycled.

Used dental appliances may be considered a biohazard. Follow your country-specific laws, directives, standards and guidelines for the disposal of used devices.

 Manufacturer:
Dentsply Sirona Orthodontics Inc.
7290 26th Court East
Sarasota, FL 34243 USA

EC REP
Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

CH REP
Maillefer Instruments
Holding Sàrl
Chemin du verger 3
CH-1338 Ballaigues

MKT-DFU0034-ML Rev. 06 Issued 03-2024

©2024 Dentsply Sirona

Biostar and Drufomat are not trademarks of Dentsply Sirona.

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