

RELIANCE ORTHODONTICS INTRODUCES


First Ever Orthodontic No Mix Self Etch Primer

ULTRA S.E.P. features a single liquid application – making effective tooth preparation quick, simple, and error-free.

- ✓ **EFFECTIVE AND EFFICIENT TOOTH PREP**
- ✓ **REDUCES CHAIR TIME WITH SIMPLIFIED PROCESS**
 - Scrub onto enamel for 5 seconds
 - Air dry
 - Apply Assure Plus (optional)
 - Air dry
 - Deliver bracket and cure
- ✓ **SINGLE LIQUID APPLICATION NO MIXING**
- ✓ **GREAT FOR BOTH INITIAL BONDING AND EMERGENCY REPAIR APPOINTMENTS**
- ✓ **EFFECTIVE FOR ALL ENAMEL SURFACES**
 - Bracket and Tube Bonding
 - Buildups
 - Increasing band cement retention
 - Attachment delivery




ONLY FROM  **RELIANCE®**

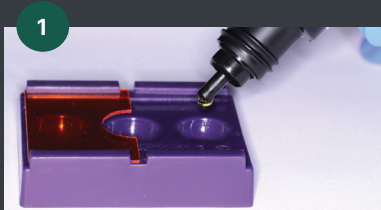


1 Dispense Ultra S.E.P. into chairside well, no mixing required




1 OR Dispense Ultra S.E.P. into Reliance dual ring wells. No mixing required

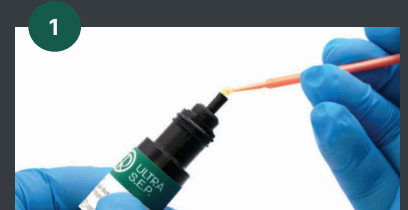

1 OR Drastically reduce material cost on REBONDS by saturating a microbrush with a single-tooth vertical dispense

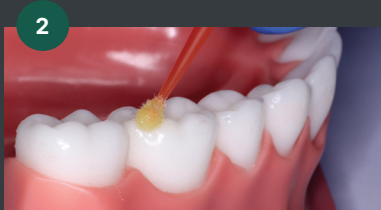

2 With a micro-brush, aggressively scrub Ultra S.E.P. onto tooth for 5 seconds

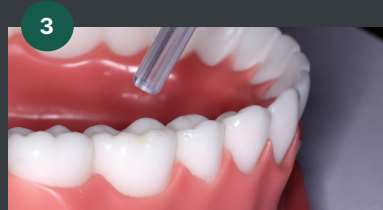

3 Air dry for 2-3 seconds before placing attachment/appliance



ASSURE PLUS

Increase any S.E.P. strength with Assure Plus*

*Study protocols and results available at RelianceU.tech