

**MONITEX**

# ShinMAX

## GS350

### Alginate Mixer

Smooth mixture in seconds for a quick and accurate impression

Patented anti-vibration mechanisms

Stylish and compact design

Reduce mixing time from minutes to seconds



# ShinMAX GS350 Alginate Mixer

The fully automated ShinMAX alginate mixer is essential to clinics who want to consistently have the smoothest homogenous alginate mixtures to take accurate impressions with. This compact and stylish mixer comes with patented safety microswitches and anti-vibration mechanisms for high quality and safe operation.



Simple steps to creating a perfect impression:



**1. Add alginate powder and water into the mixing cup.**



**2. Moisten with the stirrer for a few seconds and close the mixing cup.**



**3. Put the mixing cup into the mixing well.**



**4. Cover the lid and press "START".**




**5. After mixing, use the spatula and remove the mixed impression.**

### 3 Default Mixing Modes

Scoops	Mixing Time
☪	8 secs
☪ ☪	10 secs
☪ ☪ ☪	12 secs

### Specifications:

- Timer Range 1 ~ 16 seconds
- Mixing Modes 3 (Default: 8, 10, 12 secs)
- Speed 3,400 rpm
- Power Input AC 110 or 230V, 50/60 Hz
- Dimensions 250(L) x 205(W) x 290(H) mm
- Net Weight 17.5 kg
- Accessories Spatula/Stirrer/Scoop/Accessory Holder/  
Water Dosing Bottle/Mixing Cups(x4)

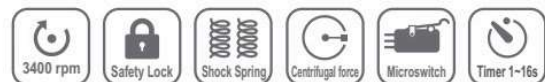

**MONITEX** INDUSTRIAL CO., LTD

5F, 70, Sec. 1, Guangfu Rd., Sanchong Dist., New Taipei City 24158, TAIWAN  
 TEL: + 886 2 29993869 FAX: + 886 2 29993647  
 Email: [monitex@monitex.com.tw](mailto:monitex@monitex.com.tw)  
[www.monitex.com.tw](http://www.monitex.com.tw)

