HASZNÁLATI ÚTMUTATÓ
USER GUIDE
MODE D'EMPLOI
INSTRUKCJA UŻYCIA
ISTRUZIONI PER L'USO DI

U Concept
The Ultimate Bio Trainer

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USER GUIDE U Concept® THE ULTIMATE BIO TRAINER

Your dentist or orthodontist, has chosen then personalised a **U Concept®** device for you.

This medical device, classified as risk class I, is the result of scientific research and development and its purpose is to make your treatment a success. It is suitable for medical use and made of soft silicone to ensure biocompatibility.

The U Concept® functional education device has been developed for patients in need of orthodontic treatment. The device helps children and young adults to relieve existing excessive muscle tension, promotes the strengthening of muscles, and assists in the correction of existing bad habits, such as mouth breathing, tongue swallowing, low tongue position, positioning of the lower lip behind the upper front teeth, and certain types of speech defects.

Thanks to its material, the device is easy to get used to and comfortable to wear, while it moves your teeth and positions your jawbones in the correct position.

Contraindication: Allergic reactions that occur while wearing the device should be reported to the treating physician and the wearing of the device should be temporarily discontinued. Gingivitis and gingival atrophy must be corrected before using the device.

Your treating physician will give you the device with personalised instructions for use. As a reminder, the following are the basic instructions for use and maintenance:

1. U Concept® devices help you get used to switching from breathing through your mouth to breathing through your nose. To make this easier, always blow

your nose before using the device. Once you have placed the device in your mouth, breathe through your nose and close your lips.

2. Inserting the trainer in your mouth is very easy. The upper part of the device has a small notch in the middle. Holding this part in the middle, place the device on your upper tooth arch, then bite into the lower part with your lower teeth.
3. When swallowing, place your tongue on the floor of your mouth with the help of the device, and keep your lips closed.

DAILY AND WEEKLY CLEANING OF THE DEVICE:

- **Daily:** Wash your device every morning with a toothbrush and toothpaste.
- **Weekly:** Clean your device once a week in hot water for 10 minutes.

WEAR TIME:

For the best results, it is essential that you follow the instructions of your treating physician and their team. They will help you get used to the device as easily as possible and teach you exercises to increase the efficiency of your device.

Basically, the general rule is that the **U Concept®** device should be worn for 12 to 14 hours a day. This means that it should be worn for 2 to 4 hours during the day and all night. The device can be worn during the day while watching TV, using your PC, reading, or doing homework ...

If you have been breathing through your mouth for a very long time instead of practising proper nasal breathing, there is a chance that the device shall fall out of your mouth during the first few nights. In order to speed up the switch to nasal breathing and keeping the device in your mouth all night long, practice the exercises prescribed by your treating physician or their assistants.

Your commitment and diligence is the ultimate key to your success. Your treating physician and medical staff will help you overcome the difficulties, so you need to wear the device and do the exercises prescribed for you.

If you follow the instructions provided, you will get used to the device very quickly and this will reduce the time of fixed device treatment.

If you notice any cracks, tears, or any other changes while wearing the device, stop using it and notify your treating physician immediately.

Serious incidents involving the device must be reported to the manufacturer and to the competent authority of the Member State of the place of residence of the user and/or the patient.

Order code	Trade name and specification
------------	------------------------------

SDUK	U Kiddy
-------------	----------------

Assists in stopping the use of pacifiers or thumb-sucking. Treatment of open bite. Treatment of cross-bite and other asymmetries. Prevention and correction of ingrained bad habits. Midline correction.

SDUS	U Start
-------------	----------------

Correction of bad habit. Treatment of distal bite. In case of cross-bite (transverse dimension checking). Treatment of open bite. Orthopaedic correction.

SDUT	U Trainer
-------------	------------------

Mixed dentition. Correction of bad habit. Neutralisation of functional matrix. TMJ release. Preliminary alignment of teeth.

EN

8

Order code	Trade name and specification
SDUTS	U Trainer Soft Mixed dentition. Correction of bad habit. Neutralisation of functional matrix. TMJ release. Preliminary alignment of teeth.
SDUC2	U Class 2 Treatment of overjet greater than 6 mm. Treatment of open bite.
SDUC2Pre	U Class 2 Pre Treatment of overjet greater than 9 mm. Treatment of open bite.
SDUC3	U Class 3
SDUC3S	U Class 3 Small Front cross-bite cases. Angle Class II cases. Promotes the growth of premaxilla. Prevents mandibular protrusion.
SDUB	U Brace
SDUBS	U Brace Small Prior to and during fixed brace treatment. Ideal during the levelling phase. Suitable for both children and adults and can be used in mixed and permanent dentition. Helps reshape the arch. Appropriate protection for soft tissues during fixed brace treatment.
SDUP1-7	U Pilot 1-7 Resolution of minimal crowding in mixed and permanent dentition. Helps after fixed brace treatment to level the potential relapse that has developed in the front region.

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